EN - Villacryl Thermo Iso-clear

Light curing varnish

The characteristics of the product: Material isolating plaster –thermoplastic material

Usage: Apply the isolator to a dry model with the help of a brush, and then cure it in a UV dental curing light. Do not apply many layers of the isolator due to making too thick a layer between the model and the denture, which can result in a lack of retention in the basal seat area.

A possible yellow discoloration of the isolator is a normal phenomenon; and it does not affect the material's quality.

Ingredients: Ethoxylated Trimethylolpropane Triacrylate, Pentaerythritol tetra (3 mercap-topropionate), Diphenyl (2,4,6- trimethylbenzoyl) phosphine oxide

Hazards and counteracting

H317 May cause an allergic skin reaction

H319 Causes eye irritation

H411 Toxic to aquatic life with long lasting effects.

P273 Do not release into the environment (avoid release to the environment.)

P391 Collect spillage.

P261 Avoid breathing dust/fume/gas/mist/vapours/spray.

P271 Use only outdoors or in a well-ventilated area.

P280 Wear protective gloves/protective clothing/eye protection/face protection.

P501 Dispose of contents/container to an authorized waste recipient.

P302 + P352 IF ON SKIN: Wash with plenty of soap and water.

P333 + P313 If skin irritation or rash occurs: Get medical advice/attention.

P305+P351+P338 IN CASE OF EYE CONTACT: Rinse cautiously with water for several

minutes. Remove contact lenses, if present and easy to do. Continue rinsing.

P337+P313 If eye irritation persists: Get medical advice/attention.

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